



**Morrison Medical
Economy Best Strap™
Instructions for use**

ENGLISH

Important!
*Read all instructions thoroughly
before using the
Morrison Economy Best Strap™*

Only qualified and properly trained personnel should use the Morrison Medical Economy Best Strap™. Any persons using this strap should practice application and be thoroughly familiar with all medical protocols and aspects of its use **BEFORE** using. These instructions are provided as a guide only. Use protocols and techniques established by your local medical control.

1

Description/Intended Use:

The Morrison Medical Economy Best Strap™ is a strap system comprised of 4 body and 2 over the shoulder straps with Plastic Side Release Buckles. Designed to secure a patients body at the shoulder, chest, hips, thighs and ankle while laying on a backboard.

Warning:

**CHECK PATIENT FREQUENTLY!
IMPROPER USE COULD RESULT IN SERIOUS INJURY OR DEATH.**

Inspect strap before each use. Destroy and discard if you observe:

- Broken stitching
- Frayed or cut fabric, straps or broken buckles.
- Other signs of deterioration

Use of products with these conditions could result in product failure resulting in serious injury or death.

2

Make sure straps are not twisted.

Make certain the strap is not too tight or too loose. You should be able to place four fingers (with your hand flat) between the patient and the strap.

Caution: Begin with ankle straps to assure proper placement of shoulder straps and assure adequate circulation.

1. Make sure the patient is placed correctly on the board.
2. Place the "Economy Best Strap™" along the patients body so that the shoulder straps (labeled Shoulder Front) lie across both shoulders and the center strap lies along anterior midline. Do not secure shoulder straps at this time.
3. Starting at one side of the ankle strap (labeled Ankle Front); thread the male end of the strap through the hand holes of the backboard to secure to the female end of the strap. Repeat for other side of this strap. Always keep Anchor strap centered.
4. Repeat the above process with the 3 remaining body straps; thigh, hip and chest area.
5. Finish with the two shoulder straps; thread the male end of the strap through the hand holes of the backboard to secure to the female end of the strap. Pull firmly to adjust. Repeat for other side of this strap. Always keep anchor strap center.

3

To Release

Begin with shoulder straps and press side of closure, pull apart to release.

Continue until all straps are separated.

Cleaning

Machine wash, air dry. Do not exceed 108°F. Do not use chlorine bleach. Washing inside laundry bag recommended.

Manufactured by:
Morrison Medical
3735 Paragon Drive
Columbus, Ohio 43228



4