

## The Linebacker® Tapeless Intravenous Tube Restraint

Unprecedented in patient comfort, unparalleled in health care provider safety and unyielding in strength. Replacing decades old methods of IV securement with speed and cost effective attributes making Linebacker® your front line choice.

### *Linebacker® – The Superior Choice*

- Latex-Free.
- Painless application and removal.
- Quick and easy to apply.
- Stronger, more secure hold than tape.
- Eliminates risk of skin infection as seen with use of tape.
- Won't damage fragile skin or cause allergic reactions due to adhesive (vs. tape).
- Won't stick to or tear gloves (vs. tape).
- Ideal for use in emergency situations when skin may be wet, bloody, sandy, etc. (vs. tape).
- Can be used on hand, arm or foot.
- Single patient use, disposable.
- Can be modified to secure other catheters such as intra-aortic balloon pumps.
- Works well on arterial lines as well as IVs.



### Applying Linebacker®:

**Step 1:** Initiate IV

**Step 2:** Peel back protective film on the Linebacker®

**Step 3:** Place IV tubing on adhesive and fold over

**Step 4:** Wrap strap around limb and secure

Visit [www.morrisonmed.com](http://www.morrisonmed.com) for demonstration video.

### Also Available: Linebacker® IV Start Kit

- 1 - Linebacker®
- 1 - IV Stabilization Dressing
- 1 - Alcohol Prep Pad
- 2 - 2" x 2" Gauze Sponge
- 1 - Iodine PVP Wipe
- 1 - Patient Label

*For more information, call 800-438-6677,  
or visit [www.MorrisonMed.com](http://www.MorrisonMed.com).*

