



3735 Paragon Drive, Columbus, Ohio 43228
P: 800.438.6677 | F: 877.469.9696
www.morrisonmed.com

SPECTRUM PELVIC IMMOBILIZER - INSTRUCTIONS FOR USE

DESCRIPTION

The Morrison Medical Spectrum Pelvic Immobilizer is an extra wide, 12" pelvic immobilization strap. It improves patient stabilization and comfort.

INSTRUCTIONS

Feed hook and loop straps through hand holes of backboard. Lay one flap over patients pelvic area. Wrap the other flap on top of the first flap as tight as possible. Only qualified and properly trained personnel should use the Morrison Medical Spectrum Pelvic Immobilizer. Any persons using the Spectrum Pelvic Immobilizer should practice application and be thoroughly familiar with all medical protocols and aspects of its use BEFORE using. These instructions are provided as a guide only. Use protocols and techniques established by your local medical control.

ITEM 1374XOR