



3735 Paragon Drive
Columbus, Ohio 43228
800.438.6677 * 877.469.9696 Fax
www.morrisonmed.com

Spectrum Pelvic Immobilizer Instructions for Use

Description/Intended Use

The Morrison Medical Spectrum Pelvic Immobilizer is an extra wide, 12" pelvic strap. It improves patient stabilization and comfort.

Instructions

Loop Velcro straps through hand holes of backboard. Lay one flap over patients injured pelvic area. Wrap the other flap on top of the first flap as tight as possible.

Only qualified and properly trained personnel should use the Morrison Medical Spectrum Pelvic Immobilizer. Any persons using the Spectrum Pelvic Immobilizer should practice application and be thoroughly familiar with all medical protocols and aspects of its use **BEFORE** using. These instructions are provided as a guide only. Use protocols and techniques established by your local medical control.